

**Operating & Safety Manual**  
**For Patient/Nurse Call System and Voice Box**

**IMPORTANT:**

Prior to using the call device and voice box, read and understand all operating instructions detailed below. Also read and understand all safe operating procedures and safety precautions outlined below. Follow these instructions and procedures as applicable.

In addition, the operator should adhere to the general safety principles and accident preventative measures as outlined by the equipment owner or other prevailing authority.

**PURPOSE:**

The purpose of the device is to supplement to current Executone nurse call system by supporting the call device and to create a user friendly push button device that produces audible patient requests.

**SPECIAL FEATURES:**

Unlike other call devices, this one is held in place by a stand that sits at the side of the patient's bed. This prevents the call device cord from becoming entangled around the patients and allows it to be easily accessed. The device is portable, so it can be moved from room to room as an option to accommodate the patient that benefits the most. It also creates audible requests with the push of a button, and it has large words and symbols that are easy to read. The voice box offers the option to communicate to the nurse in either a male or female voice.

**SPECIFICATIONS:**

Voice Box (with Exutone holder):

Dimensions: 18" x 6" x 3" (width x height x depth)

Weight (with Exutone in its holder): 8 lbs

Stand:

Height: 3' to 5' adjustable

Reach: 6" to 18" adjustable

Weight: 20 lbs

Power: 9v DC wall adapter

Standby Current (when activated): 125 mA

In use Current: 200 mA

## **OPERATING INSTRUCTIONS:**

### **Nurse Instructions:**

- 1) Use fasteners on stand to adjust position for easiest use by the patient. (Turn counterclockwise to loosen and clockwise to tighten.)
- 2) Make sure all fasteners are completely tight and legs are in their widest position before placing in front of patient.
- 3) Place Executone call device in its holder.
- 4) Place device over the bed in front of the patient.
- 5) Plug in power adapter to the back of the voice box and use plastic clips to connect power cord to Executone cord.
- 6) Select male or female voice using switch on the left side of the device.
- 7) After responding to a call, deactivate system by pressing red button on the back of the voice box.
- 8)

### **Patient Instructions:**

- 1) Press the red "Nurse" call button to activate the call device and voice box.
- 2) Wait to hear the activation message and the nurse reply.
- 3) Select the button(s) that best describe your situation.
- 4) Wait for the nurse to arrive and deactivate the call device and voice box.

## **SAFETY PRECAUTIONS:**

Prior to use, make sure that the stand is properly secured and all fasteners are tight before allowing the patient to use the device.

1. Position legs in their widest, most stable position to avoid tipping.
2. Do not place any of the device cords around the patient or bed frame.
3. Always clip power and Executone cord together to avoid excess wire hanging.
4. Do not place any objects (especially food or liquids) on the device or stand.
5. Avoid spilling any liquid on the device.

## **MAINTANANCE INSTRUCTIONS:**

Wipe off any excess moisture on the device with a dry towel.

Wipe off dust from the device at least once a week. DO NOT use liquid cleaner.

## **TROUBLESHOOTING:**

- **Under no circumstances should you attempt to open the device unless instructed to do so by the manufacturer.**
- If the device is not functioning properly, unplug the device and plug it back into the power outlet.
- If the device continues to not function properly, switch the voice selection switch from the current setting to the other gender setting.
- If the device continues to not function properly, contact the manufacturer for further details.